420325A

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EN SOFT-ADHERENT DRESSING WITH POLY-ABSORBENT FIBRES

PRESENTATION

The **UrgoClean** dressing, an innovative product from URGO, is presented as a sterile, non-woven pad of highly absorbent and cohesive poly-absorbent fibres (polyacrylate).

This innovation uses thermobonding to bond polyacrylate fibres that are, arranged in parallel to the surface of the wound in a structured pattern, which give **UrgoClean** its specific properties of absorbing and trapping slough residues from the wound, as well as its resistance to traction.

The poly-absorbent fibres pad is coated with a soft-adherent TLC layer (Technology Lipido-Colloid) which gels easily, thereby aiding the drainage of slough residues and making it easier to apply and remove the dressing.

MODE OF ACTION

- On contact with wound exudate, the lipido-colloid particles from the TLC layer form a gel, creating a moist environment favourable to wound desloughing.
- At the same time, the poly-absorbent polyacrylate fibres gel and bind to the slough residues, absorbing and draining it to aid elimination (autolytic debridement).
- This binding and drainage of slough residues by **UrgoClean**'s poly-absorbent structure combined with its TLC layer therefore promote the debridement phase in sloughy and exuding wounds.

PROPERTIES

- Absorption of exudate, gelling and drainage of sloughy residues during the debridement phase.
- Absorption and trapping of bacteria by the structure of poly-absorbent polyacrylate fibres.
- Maintenance of a moist environment in contact with the wound, promoting healing of debrided zones.
- For the management of minor bleedings.
- Removal of the dressing in one piece due to the resistance to traction of the poly-absorbent fibres.
- Painless removal with no trauma to the wound and protection of surrounding skin due to the soft-adherent TLC layer.

INDICATIONS

UrgoClean is indicated for the desloughing phase of chronic exuding wounds (leg ulcers, pressure ulcers, diabetic foot ulcers) and potentially sloughy wounds: acute wounds (burns, skin abrasions, traumatic wounds), post-operative wounds, cancerous wounds.

METHOD OF USE

1 – Wound preparation:

- Clean the wound according to local protocol, then rinse with normal saline. If an antiseptic has previously been used, rinse the wound carefully with normal saline before applying **UrgoClean**.

- The use of **UrgoClean** does not dispense with the need for mechanical debridement where required.

2 – Dressing application:

- Remove the protective tabs.

- Apply the soft-adherent side of **UrgoClean** to the wound and its contours. **UrgoClean** can be cut using sterile scissors to adjust the size of the dressing to the wound if necessary.

- Cover **UrgoClean** with a secondary dressing suitable for the location of the wound and level of exudate.

3 – Dressing changes:

UrgoClean dressings should be changed every 1 to 2 days during the wound desloughing phase, then as often as required (up to 7 days) depending on the exudate volume and the clinical condition of the wound. Discard any unused parts of the dressing according to local procedures.

WARNINGS AND PRECAUTIONS FOR USE

- The soft-adherent TLC layer of **UrgoClean** adheres to latex surgical gloves. It is therefore recommended that the dressing be handled carefully, avoiding any contact with the soft-adherent side or using sterile forceps.

- The concomitant use of a cream, lotion, ointment or emulsion is not recommended.

- Due to the non-occlusive nature of this dressing, **UrgoClean** can be used on infected wounds under close medical supervision.

- During the desloughing process, the wound may appear to be larger in size. This is due to the effective removal of slough.

- **UrgoClean** must not be used in a hyperbaric chamber.

- Do not re-sterilise the dressing.

- (*) Check that the sterile protector is intact before use. Do not use if package is damaged.

- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.

CONTRA-INDICATIONS

- Do not use if there is a known sensitivity to **UrgoClean**.

- Do not use **UrgoClean** as a surgical sponge for heavily bleeding wounds.

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